



Open the sterile packaging of the jar and the vial as well as the spatula.



Slightly tap on the lid of the jar or gently tap the bottom of the jar against a rigid surface in order to retrieve all the powder.



Transfer the entire content of the vial into the jar by inversion of the vial.



Perform a rotary motion with a spatula for **2 minutes** to mix the powder and liquid phases together and obtain a homogeneous cement.



Apply the cement at the surgical site with a spatula.

PREPARATION

INJECTION WITH THE SYRINGE (Optional)



Use a spatula to transfer the cement into a 5 or 10 ml sterile syringe (not provided).



Slowly apply pressure to the pushrod to push the cement forward into the syringe to dispense the BIOCERA-VET product.